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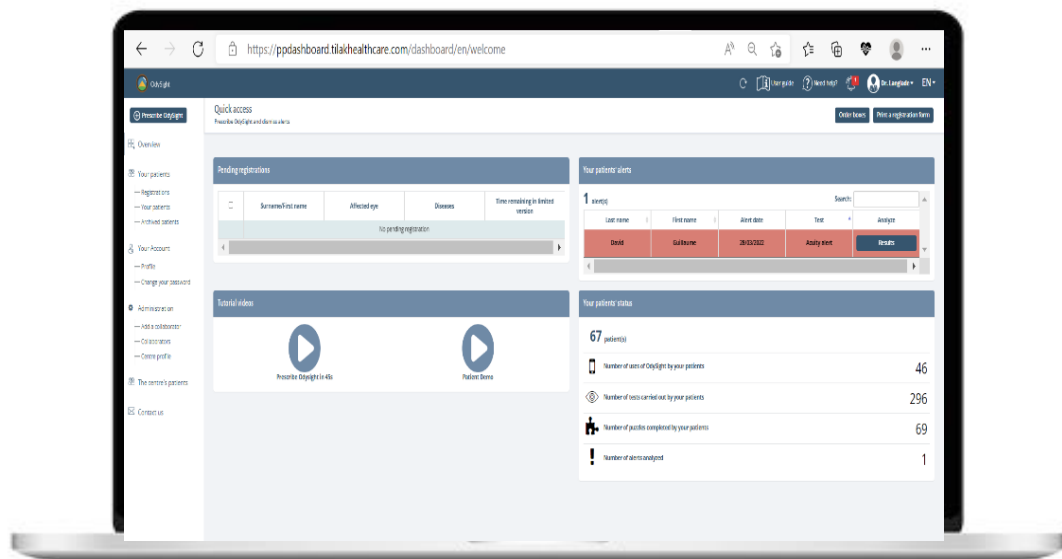
CE marking for European compliance. The application complies with the European Union Directive on medical devices.

<i>OdySight presentation</i>	3
Intended use	3
Product presentation	3
Contre-indications	3
Product description	4
Product performance	5
Security	7
<i>General set up and use</i>	7
Creating your Center account	7
Creating a collaborator account	7
User permissions	8
Access your OdySight Account	9
Navigation	10
Home Page	10
Navigation Menu	10
Navigation bar	10
Patient List.....	10
Archived Patients	12
Results Page	12
User Profile Page.....	12
How to prescribe OdySight	13
View Results	13
Visual Acuity Decreases notifications	13
Identifying decreases in Visual Acuity	13
Qualify a decrease in Visual Acuity.....	14
Log out of the Dashboard	14
Updating the Dashboard	14
<i>Technical information</i>	14
Hardware	14
Security	14
<i>Warnings and precautions</i>	15
<i>Contact us</i>	16
<i>Symbols used on the device labelling and user guide</i>	16

OdySight presentation

Intended use

OdySight® is a medical software intended for remote monitoring of the visual parameters: near visual acuity and symptoms such as metamorphopsia and scotoma. The test results of user's self-evaluation are transmitted to the physician.

OdySight® doesn't interpret the test results and it is not intended to diagnose but rather to help patients determine if they need to consult a physician for diagnosis and treatment.

Indications: OdySight® is indicated for adult patients aged 18 and over.

Product presentation

OdySight consists of an app for the user and a dashboard for the physician and Healthcare Professionals.

OdySight could be recommended for users that, from the point of view of the ophthalmologist or other physician, the user will benefit from remote monitoring of visual parameters: near visual acuity or the evaluation of metamorphopsia and scotomas.

OdySight could be recommended for patients requiring regular, close monitoring of vision. It is to assist the physicians with the patient's visual acuity follow up care between scheduled clinic visits.

The app is prescribed to the patient by a healthcare professional.

OdySight doesn't interpret the test results and is not intended to diagnose. The diagnosis and patient treatment remain the responsibility of the physician.

OdySight does not replace scheduled clinic visits.

The physician prescribing the device must ensure that the patient is competent to use the device.

OdySight shall be prescribed to adult users who have:

- a) Binocular visual acuity with habitual correction equal to or better than 3/10 (0,5 logMAR; (20/63))
- b) Access to a compatible smartphone and/or tablet
- c) A valid email address and phone number to create an account
- d) The competence to use the app and complete the test modules remotely, without the guidance of a healthcare professional
- e) The capability to regularly perform a simple self-test at home

Contre-indications

OdySight app is contra-indicated for:

- epileptic users
- binocular visual acuity with habitual correction less than 3/10 (0,5 logMAR; (20/63))

Product description

OdySight is a digital system made up of two components: a patient-facing application and a dashboard designed for healthcare professionals.

The OdySight application comprises several modules:

- the Near Visual Acuity module, enabling users to test their near vision; based on the ETDRS (Early Treatment Diabetic Retinopathy Study);
- the Amsler Grid module, allowing users to assess metamorphopsia and scotomas; based on the standard paper version of the Amsler Grid;
- the Authentication module, allowing users to log in securely to the OdySight application;
- the navigation menu, which allows users to move through the OdySight application;
- the game and DLC (downloadable content) module, referring to puzzles designed to support user engagement and retention.

Note: the game module is not part of the OdySight medical device.

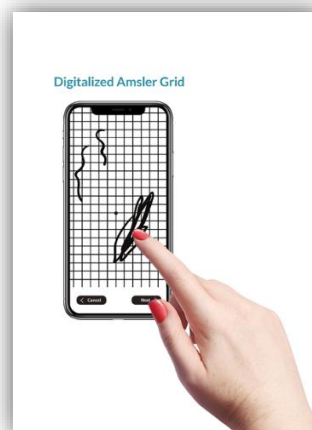
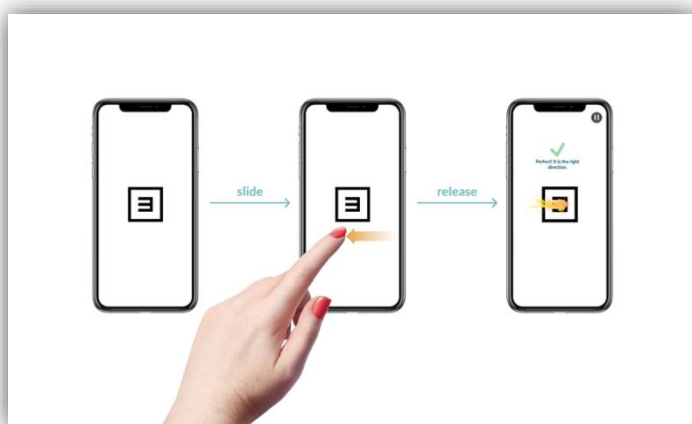
The application is available by prescription only and can be used on supported iOS/Android smartphones that meet the minimum technical requirements (see section 3.1).

- OdySight app

The app consists of two parts: vision tests (visual acuity, Amsler grid), and a game (puzzles). The vision tests are intended to collect test results similar to those usually obtained in a medical center using standard methods.

The visual acuity test assesses near vision, while the Amsler grid test assesses the evolution of scotomas (black spots) and / or metamorphopsia (deformation).

Near visual acuity and Amsler Grid tests in the OdySight app

- OdySight Dashboard

The dashboard, intended for use by healthcare professionals, is a secure website for viewing the results of vision tests taken by their patients.

Product performance

OdySight allows the remote monitoring of visual parameters.

OdySight is a tool intended to complement the standard medical monitoring of your patients.

The application does not replace prescribed medical visits or examinations, which patients must continue to undergo.

If the results of the medical tests seem to indicate the need, OdySight will advise you to contact your patient.

The visual tests available in OdySight have been evaluated and compared with standard tests performed in medical centers during a clinical trial. The clinical study demonstrated that the visual acuity tests and the Amsler Grid provided in OdySight showed a strong concordance with tests performed in medical settings (standard methods used – the ETDRS scale for distance and near visual acuity, and the paper-based Amsler Grid).

Each day, OdySight informs patients which eye needs to be tested. On average, an OdySight test takes 5 minutes to complete.

If the test results suggest the need for further attention, OdySight will advise patients to contact you directly through the application.

The tests allow patients to unlock access to puzzles. Game sessions are relatively short so as not to cause eye strain.

OdySight visual acuity alerts are sent in the event of a decrease in visual acuity. These notifications are delivered both to the patient and to the ophthalmologist to facilitate follow-up. They support and assist the ophthalmologist in taking action: contacting the patient and scheduling an appointment to clinically verify visual acuity. OdySight is a complementary tool for physicians, enabling them to monitor their patients' visual acuity between scheduled medical visits.

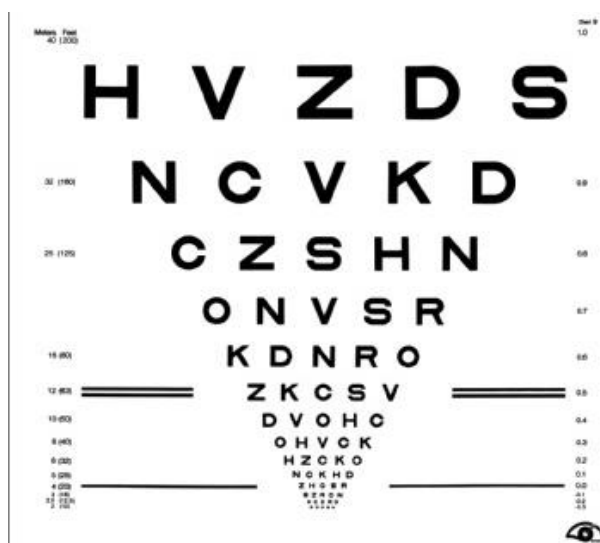
The diagnosis remains the responsibility of the prescribing physician.

VISUAL ACUITY TEST

Visual acuity is the ability to discern the thinnest possible character with maximum contrast. In the monitoring and diagnosis of ocular diseases, we carry out this measure eye by eye using optimal optical correction (corrective lenses).

The international standard for vision tests is the ETDRS. The test used in OdySight is directly inspired by this reference.

For reasons of international compatibility, OdySight uses the version of the ETDRS with the "Tumbling E", which is already used in certain sight tests:

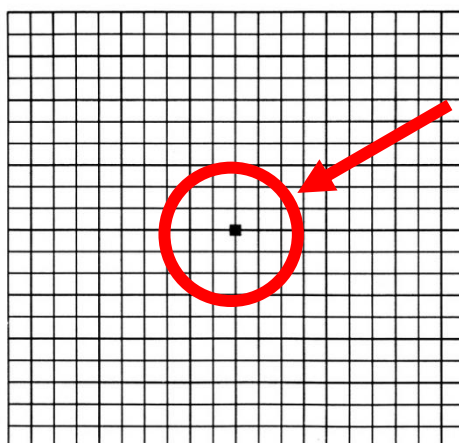
Near visual acuity, as calculated in OdySight, is tested at a set distance of 40 cm, one eye at a time, with glasses for near vision.

IMPORTANT: If the results of your patient's tests require it, OdySight will advise you to contact your patient.


AMSLER GRID TEST

The Amsler grid was created by Marc Amsler, a Swiss ophthalmologist, in 1945. It is a self-monitoring tool for maculopathies such as AMD and macula edema.


By staring at the center of the grid with one eye open at a set distance (around 40 cm), a person may see metamorphopsia (wavy lines) or scotoma (black patches) appear, which are symptoms of a change to an optical condition. In this case, monitoring by an ophthalmologist is required

OdySight Dashboard v2

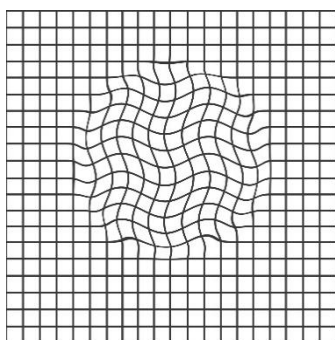Distorted lines

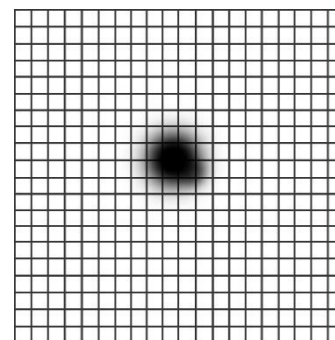Dark spot

The Amsler Grid does not send notifications advising patients to contact their physician, as may be the case with the visual acuity test. By comparing the different results obtained, if you notice the appearance or worsening of any such anomalies, please contact your patient.

Security

Patients' health data can be accessed at any time by the ophthalmologist through the online dashboard, which access is both secure and restricted.

Personal health data are made available to the physician on a dashboard. Such data including the results of the vision tests are stored in the U.S. The transmission of data by the server to the dashboard is secured and encrypted.

General set up and use

The dashboard allows you to prescribe OdySight and to follow the evolution of your patients' visual parameters obtained during their vision tests performed with the application.

Creating your Center account

To create a Center account, you must fill out a center creation form and then click on the confirmation button.

Once the form is submitted, you will be redirected to an account creation form. Fill it in and click on the confirmation button.

Once the form is submitted you will receive an automatic email containing an activation link. Click on the activation link to confirm the creation of your account. Once this action is completed, your account will be created, and you will be able to access your dashboard.

Creating a collaborator account

Account creation on the dashboard can be carried out in two ways.

1. First option: a user creates their own account and must then create their center. Alternatively, a user may create an account independently from the account creation page. After the welcome screen, the user is invited to create a center and select their role (see section 2.3 "User Rights"). They must then enter and validate their email address via an automatically generated verification link. The next step is to complete the profile information. For physicians practicing in France only, it is sufficient for the user to enter their RPPS number and AM number so that their information is automatically retrieved via the Annuaire Santé (Health Directory). Once the profile information has been entered, the user is prompted to provide and verify their phone number by entering an SMS verification code before setting their password. The account is finally created once the user has clicked the "Confirm" button.
 2. Alternatively, the user may be prompted to join an existing center by an existing user. In this case, the user receives an invitation by email and must create their account without having to set up a new center.
To invite a collaborator to join their center, a user must go to the "Administration" menu and select "Add collaborator." The user then simply completes the collaborator addition form with the collaborator's email address, role, and administrative permissions, and clicks the "Confirm" button.
Once the form has been completed, the added staff member receives an automated email inviting them to join the existing center. The email includes a registration link. By clicking this link, they will be redirected to an account registration form. Once completed, their account will be created and linked to the existing center.
-  Only an administrator can invite a staff member to join their center.
-  If any required fields (marked with an asterisk *) are incorrectly filled during the registration process, an error message will be displayed.

User permissions

Each healthcare professional account is assigned a role at the time of creation. . This role determines which features of the dashboard the user can access. Some sections and actions may be restricted depending on the user's access rights. A detailed explanation of user permissions is available in our FAQ at: <https://www.odysight.fr/en/espace-pro/>

When adding a new team member, four different roles can be assigned — each with specific access levels. All users within a center are connected under the same structure.

Physician:

The physician is one of the medical doctors in the center. They have full rights: They can prescribe OdySight, monitor and evaluate any decrease in near visual acuity, and access both patient profiles and test results.

Physicians can be linked to both medical (other physicians and orthoptists) and non-medical staff (IT and administrative staff).

Orthoptist:

Orthoptists must be affiliated with a doctor and can only access patient profiles and view test results. They cannot prescribe OdySight or diagnose visual acuity loss.

Secretary:

Secretaries only have access to patient profiles. They cannot prescribe the application, assess visual acuity loss or view patient test results. Like medical staff, they must be affiliated with a doctor.

Technical team (IT):

Technical teams can only modify centre information and invite new employees. They cannot prescribe the application or access patient profiles.

Two additional rights can be assigned to each employee:

- The 'administrator' right allows them to modify centre information and invite other employees.
- The 'access to all patients' right allows them to view the profiles of all patients at the centre, regardless of their assigned doctor.

Access your OdySight Account

The OdySight dashboard is accessible at the following address:

<https://dashboard.tilakhealthcare.com/dashboard/en/login>

To access the OdySight dashboard, follow these steps:

- Go to the dashboard login page.
- Enter your email address and the personal password you created during account setup in the corresponding login fields.
- A temporary code valid for one hour will be sent to you by email or SMS, depending on your account settings. Enter this code in the designated field and click "Validate" to access your dashboard.

You may check the box "Remember this device as a trusted device" to avoid entering a temporary code during future logins.

This is a two-factor authentication process (i.e., with two codes), designed to ensure the security of medical data.

If you lose your password, follow the steps below:

1. Go to the login page of the dashboard and click on "Forgot your password?"
2. Fill in your email address in the dedicated field. A message will confirm that a password reset email will be sent to you.
3. Check your mailbox and click on the reset link.
4. Choose a new password (*for more information, see section 3.2 "Security"*) and type it twice.
5. Click "Save".

You will now be able to access your Dashboard.

For physicians based in France, the dashboard is also accessible via the ProSanté Connect authentication service. By clicking the ProSanté Connect logo, you will be redirected to their secure login portal. Once authentication is completed, you will automatically return to the OdySight dashboard.

Navigation

Home Page

The home page is the first page that appears when you log in to the dashboard. From this page you can:

- Prescribe OdySight to a patient
- See your patients presenting a decrease in Visual Acuity.
- Check their test results, and qualify Visual Acuity decreases
- Access the Navigation menu
- Access the Navigation bar
- See patient statistics

Navigation Menu

The navigation menu is located on the left side of the dashboard. From this menu you can access:

- The Home page (Dashboard)
- The Prescription form (Add a Patient)
- OdySight patient list (Patients)
- Archived Patient list (Archived Patients)
- Your account settings (Profile and Change Password)
- The Administration menu (Add and update Healthcare Professionals)
- OdySight customer service (Contact us)

Navigation bar

The navigation bar is located at the very top of your dashboard. It includes:

- A button to return to the home page (the house)
- The User Guide (the book)
- A link to a Questions and Answers page with answers to frequently asked questions about the dashboard (the "i")
- A button to display the notifications of your patients presenting a decrease in Visual Acuity (the bell)
- A button to access your profile settings and to log out of the dashboard.

Patient List

To access the patient list, click on the "Monitored Patients" tab in the navigation panel bearing the same title.

- Patients showing a decrease in visual acuity appear in red and are listed at the top.
- Active patients are shown in green and are second on the list. A patient is considered active if they have completed at least one visual test in the last thirty days.
- Inactive patients are displayed in gray. A patient is considered inactive if they have never downloaded the app, or if they have not completed a visual test in the last 30 days.

Clicking on a patient's icon will redirect you to their profile page. From there, you can edit the patient's information or contact them by SMS — to remind them to take a new test or to provide any other instruction you deem appropriate.

You can sort the patient list based on the action you wish to perform: renew, follow up, or archive. Simply click on the corresponding column header.

You may also navigate through the following tabs:

- Patients to follow up: a list of all patients who received an OdySight prescription but have not yet downloaded the app.
- Patients requiring renewal: a list of all active or inactive patients whose prescription has expired.
- Archivable patients: a list of all your inactive patients.

From these sections, you can:

- View a patient's results by clicking on 'Results' (for more details, see the section 'The results page').
- Assess a patient's visual acuity loss by clicking on their line to display a visual acuity loss assessment form.
- Modify a patient's information by clicking on their line.
- Reach out to inactive patients (by email) by checking the box(es) for the patient(s) concerned and clicking on the 'Reach out' button. This function is only available if the patient has never downloaded the application.
- Renew the prescription for one or more patients (valid for 1 year) by clicking on the 'Renew' button via multiple selection.
- Archive one or more patients by clicking on 'Archive' via multiple selection. This function is only available if the patient does not have reduced visual acuity.
- For any renewal, archiving or follow-up action, a confirmation message will be displayed to allow you to confirm your choice.

Archived Patients

To access the list of archived patients, click on "Archived Patients" in the navigation menu. This list contains all patients that have been archived manually (from the patient profile) or automatically (after 90 days of inactivity).

From this list, you can:

- View a patient's results by clicking on "Results" (see "*The Results Page*" section for more details).
- Modify a patient's information by clicking on their profile
- Send a follow-up email to a patient by clicking on the "Restart" button. This function is only available if the patient has never taken a vision test.
- Delete a patient's profile by clicking on "Delete". This function is only available if the patient has never downloaded OdySight or has never used their encrypted key (activation code for the OdySight patient account).

The personal and medical data of an archived patient is automatically deleted after two years of inactivity.

Results Page

The results page is where you can view a patient's visual acuity chart for each eye, along with any drawings made on the Amsler grids. To access this page, click on "Monitored patients" in the navigation menu, then click the "Results" button next to the patient's name. From this page, you can:

- Qualify a decrease in Visual Acuity
- Consult visual acuity test results
- Consult Amsler grid results with previous Amsler grid drawings

User Profile Page

The user profile page allows you to view and edit your personal information.

Editing information:

You may update the following fields to define your contact preferences: email address, mobile number, phone number displayed to patients, and preferred method of receiving the login code.

The default unit of measurement when an account is created is **Monoyer**. Users can choose their preferred unit of measurement from their profile by selecting one of the four available options:

- Decimal
- LogMar
- Monoyer

- Sneller

Once selected, the unit is automatically updated and applied across the user's entire dashboard.

Only for physicians based in France, the following fields become non-editable once a valid RPPS number has been entered and confirmed: medical title, specialties, professional first name, professional last name, and RPPS number. To edit any of this information, updates must be made directly via the Annuaire Santé website.

How to prescribe OdySight

The OdySight prescription form is available from two dashboard pages:

- From the home page: click on "prescribe OdySight"
- From the navigation menu (on the left): click on "Add a patient" to be taken to the page where you can fill out the patient's information to prescribe OdySight

To prescribe OdySight, fill out one of these two forms with the required information about your patient. Then click "Create". Your patient will automatically receive an e-mail and a text message containing all the information required to use OdySight. The prescription procedure is then complete.

View Results

Your patients' medical data can be consulted at any time on your online dashboard, with restricted access.

To see patient results:

1. In the navigation menu, click "Patients". The list of all patients to whom you have prescribed OdySight will be displayed (except for archived patients).
2. Go to the patient line of your choice. In the "Actions" column, click on "Results". You will then be redirected to your patient's results page.

Visual Acuity Decreases notifications

Identifying decreases in Visual Acuity

OdySight notices you about a significant decrease in Visual Acuity when a change is observed in a patient's visual acuity results. The patient and the physician are informed as follows:

1. The patients receive an email and a text message asking them to contact their physician. A notification is also displayed in the application, asking them to confirm whether they have contacted their physician.
2. The physician receives a notification on their dashboard and a mention of the patient presenting a decrease in Visual Acuity in the bi-weekly summary email. In addition, the patient's profile appears on the home page of the dashboard, in the list of patients presenting a decrease in Visual Acuity (in red).

If a patient shows a decrease in visual acuity, no additional alert for the same eye can be triggered for at least 6 days, allowing time for the initial decline to be addressed.

Qualify a decrease in Visual Acuity

To qualify a decrease in Visual Acuity:

1. Go to the patient results page. Check the patient's most recent vision tests to see if the decrease in Visual Acuity seems to be significant.
2. If necessary, speak with your patient by phone. Make sure the decrease in visual acuity is significant. Offer the patient an appointment if necessary.
3. Then, go to your dashboard and click on the patient's name. A decrease in Visual Acuity Qualification Form will be displayed.
4. Answer the decrease in Visual Acuity Qualification Form to confirm the relevance of the decrease in visual acuity as well as any possible treatments.
5. Finally click on the "Confirm" button to definitively qualify the patient's decrease in Visual Acuity.

Once the decrease in Visual Acuity is qualified, your patient will revert to active status on your dashboard (their profile will no longer be displayed in red, but in green).

Log out of the Dashboard

When you have finished using the dashboard, it is important that you log out to protect medical data. To do so:

1. In the bar at the very top of the dashboard, click on the icon representing a physician.
2. A menu will be displayed, including the "Logout" button. Click it to log out.

Updating the Dashboard

Updates to the Dashboard will happen automatically, there is nothing specific that you need to do.

Technical information

Hardware

The dashboard is used on a web browser (Chrome, Firefox and Edge) and is available on any device (computer, tablet, or smartphone) with these web browsers. This device can be on any operating system (Windows, MacOS, Linux, iOS, Android, etc.). The use of the dashboard requires an internet connection.

Security

The results of the medical tests are stored by a certified health data hosting provider, in compliance with the requirements of European Regulation 2016/679 on data protection.

Here are some precautions to take for safety reasons:

- Do not create your password with personal information (date of birth, name, nickname, etc.).
- Do not use obvious sequences (qwerty, 1234, etc.).
- Don't put special characters at the end; don't start with uppercase (Randomword0? => ran?Dom0word).
- Never give out your password.
- If you think your account has been compromised, change your password!
- Disconnect from the dashboard manually when you are not using it.

Your account will be automatically logged out of the dashboard after 2 hours of inactivity, if you logged in using your email and password.

Only for physicians based in France, if you logged in using ProSanté Connect, your session will automatically expire after 30 minutes of inactivity.

When you leave the room where you use your computer, be sure to lock or turn off your computer to protect it from unauthorized access.

Warnings and precautions

OdySight does not replace the patients' healthcare facilities visit and tests prescribed by the healthcare professional as part of monitoring of the vision.

OdySight does not provide an interpretation of test results, diagnose or treatment recommendations. It is not intended to provide a diagnosis. Diagnosis remains the responsibility of the physician.

All medical decisions must be made by a healthcare professional.

If the patients notice a change in their vision and think there is a need, they need to contact the healthcare professional who prescribed OdySight. Their contact information is available in the app.

The app should be used only by the patient whom the application has been prescribed to ensure the reliability of the data provided to the healthcare professional.

The test results may differ from those obtained during a medical appointment. They should be considered as relative values.

The OdySight Visual acuity decrease notifications are not intended to have a direct impact or change the clinical care flow for patients suffering from a chronic ocular disease, but it is only intended to help healthcare professionals to improve patient follow-up.

Do NOT use this product to diagnose vision problems.

Tilak Healthcare is not liable for data collected by mistake on the device due to misuse or malfunction resulting from abuse, accidents, modifications or negligence.

Contact us

If you have any problems with OdySight, please see our FAQ at <https://www.odysight.fr/en/espace-pro>

If you don't find an answer to your question, contact us at: supportUS@odysight.app

Any serious incident occurring in connection with the device must be brought to the attention of the manufacturer and to the competent authority of the Member State in which the user is established.


Symbols used on the device labelling and user guide

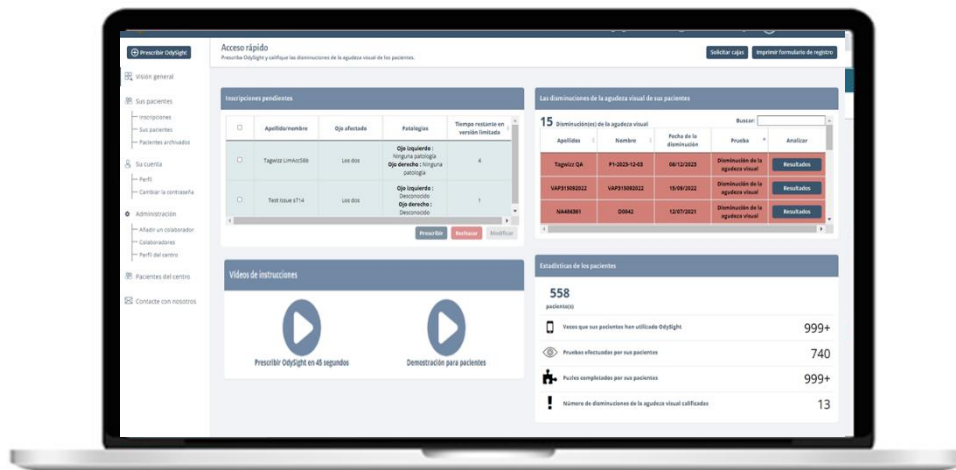	CE marking for European compliance. The application complies with the European Union Directive on medical devices.
	Fabricant
	Consult the instructions for use
	Caution/warning
	For your information

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